

The right condom fit

- The condom should fit tightly so that it does not slip off during sexual intercourse. However, rolling it over the stiff penis should not cause any pain.
- If wrinkles appear and the condom can be pushed back and forth on the stiff penis, it is too big and can slip off or tear more easily.

In addition to the slim fit, Ritex also offers condoms in standard sizes such as „RR.1“ as well as extra large „XXL“ condoms.



Slim fit with big effect

Condoms made of natural rubber latex are highly elastic and „actually always“ fit – unless they are too big. Then there is an increased risk of slipping or tearing.

With their circumference of 94 instead of 106mm, Ritex condoms „47“ offer a pleasantly firm fit and a particularly safe feeling, especially for slimmer penis sizes. The product name „47“ is derived from the width of the flat-lying condom (=half circumference).





Find the right condom size

Not sure which condom is right for you? Then simply measure it. Either print out this page (printed in DIN A6), cut out the condom gauge or simply take a tape measure, carefully wrap it around the stiff penis and read off the circumference.

- Up to 11 cm, the slim fit is optimal
- From 11 cm standard sizes are the right choice
- From 13 cm circumference Ritex XXL condoms fit best



14
13
12
11
10
9
8
7
6
5
4
3
2
1
0





RITEX QUALITY PROMISE

DERMATOLOGICALLY TESTED
VERY WELL TOLERATED BY THE SKIN

Ritex condom and lubricant range



More info under www.ritex.com